STANCARE (Wound dressing)

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◆ preparation before use

- 1) Before use, check the product packaging for damage/expiration date.
- 2) Do not use if the packaging or product is damaged.
- 3) Be sure to read the user manual before use.

◆ METHOD OF USE

- 1) Clean the area to be applied.
- 2) Apply an appropriate amount of product evenly.

◆ STORAGE AND MANAGEMENT METHOD AFTER USE

- 1) This product is disposable and must not be reused or re-sterilized.
- 2) Dispose of according to appropriate procedures after use.

◆ GENERAL PRECAUTIONS

- 1) Before use, read the product manual carefully before use.
- 2) Do not reuse or re-sterilize.
- 3) Do not use products past their expiration date.
- 4) Do not use products whose packaging has been opened or damaged.

◆ PURPOSE OF USE

1) This is a wound dressing used to protect areas where the skin barrier is damaged or destroyed, such as first-degree burns, wounds with little exudation, and dry skin.

◆ COMPLAINTS

Upon receiving any complaints or complaints regarding the quality, identity, durability, reliability, validity, and/or performance of the product, medical professionals should notify manufacturer or their local representative. Furthermore, if the instrument is malfunctioning or suspected to be malfunctioning, it should be immediately notified to manufacturer or its local representative. If products of manufacturer have been improperly operated or have caused or contributed to the patient's serious injury or death, contact its local representative or manufacturer by phone, fax, or document as soon as possible.

Please fill out the following when receiving all complaints.

-Manufacturing number of components (Lot. Number)

-Product name

-Name and address to be contacted

-Explain the cause of the complaint in detail

For additional information or complaints, please contact:

※Please check the attached label for product and manufacturer information.

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Manufacturer : Mantiz Co., LTD.